

## ROSE K2 XL™ semi-scleral lens Fitting Tips

Indication	Flatten Base	Steepen Base	Increase Diameter	Decrease Diameter	Increase Edge Lift	Decrease Edge Lift	Toric	Comments
Lens riding low	●		●		●			
Insufficient movement/ binding		●			●			
Excessive movement	●					●		
Poor visual acuity, flare or ghosting	●							Check for any significant residual astigmatism. This can be corrected on the front of the lens.
Insufficient fluorescein under the edge					●			Must be judged within 1 minute of inserting Fluorescein
Staining on highest point on the cornea		●						
Corneal staining just inside the limbus		●	●		●			Problem Insufficient movement
Tight in one meridian, loose in the opposite meridian							Toric periphery	
Loose or tight in one quadrant only								Order ACT or Reverse ACT. Available in up to two quadrants
Fluorescein edge band too wide				●		●		
Fluorescein edge band too narrow			●		●			
Ghosting or cloudy vision after several hours of wear		●	●		●			Problem insufficient tear exchange
Bubbling, orange peel stippling or dimple veiling		●	●			●		Requires better alignment with sclera. May require Quadrant Specific edge lift /ACT
Discomfort initial or ongoing	●		●			●		Discomfort is most often edge sensation. May require ACT for better alignment with sclera
Discomfort on removal		●			●			Lens too tight, not sufficient tear exchange